

Cleaning/Disinfecting of goggles and face shields

- Before leaving the isolation room, remove gloves and perform hand hygiene.
- Remove goggles and/or face shields and place them on a towel or paper towel
- Spray goggles and/or face shields with Virex
- Perform hand hygiene.
- Remove gown, rolling the outside of the gown inward to prevent contamination
- Remove mask
- Perform hand hygiene
- Take goggles/face shield, still in towel, to isolation cart
- Use hand sanitizer
- Apply fresh gloves
- Wipe down goggles/face shield and place back into cart (ensure eye protection stays wet for minimum of 3 minutes)
- Remove gloves and perform hand hygiene